JOBST®

INSTRUCTIONS FOR USE

Medical compression garments

JOBST® Confidence

- | | | |
|---|--|---|
| (EN) Medical Compression Garment | (NL) Medische compressieokous | (SL) Medicinsko kompresijsko oblacilo |
| (DE) Medizinische Kompressionsversorgung | (PT) Peça de vestuário de compressão médica | (RU) Медицинская компрессионная одежда |
| (FR) Vêtement de compression médical | (PL) Medyczna odzież uciskowa | (IS) Þrýstifatnaður í lækingaskyni |
| (ES) Prenda de compresión | (HU) Gyógyászati kompressziós ruhadarab | (AR) رداء ضاغط طبي |
| (SV) Medicinskt kompressionsplagg | (IT) Indumenti medicali compressivi | (TR) Tibbi Kompresyon Giysisi |
| (NO) Medisinsk trykbandasje | (FI) Lääkinnällinen kompressiovaate | (HE) בגד לחץ רפואי |
| (DA) Medicinsk støttestrømpe | (RO) Produs de compresie Medical | (CZ) Zdravotní Kompresní Výrobky |

FIBER CONTENT

EN FIBER CONTENT

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FR COMPOSITION

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NO FIBERINNHOLD

DA FIBERINDHOLD

NL VEZELGEHALTE

PT TEOR DE FIBRA

PL SKŁAD

HU ROSTTARTALOM

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RO CONTIN FIBRE

SL VSEBNOST VLAKEN

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Dear Customer,

Thank you for choosing JOBST®, the medical compression garment that incorporates gradient compression for the management of venous and lymphatic diseases. JOBST® offers you a wide selection of products for these indications.

1. Application

Tips for wearing

JOBST® Confidence garments:

To get the best medical result from your JOBST® Confidence garment, wear it regularly and consistently. Ideally you should put on your compression garment in the morning, immediately after getting up.

You may experience some difficulty putting on the garment in the beginning, but this will become easier after a few days of use. When applying the compression garments, please do not pull on the edge of the garments instead follow these recommendations:

Turn the arm sleeve inside out and fold the front part of the compression arm sleeve inward.

Use the other hand to pull the inside-out arm sleeve as far over the other arm as possible.

Then fold the upper end outward and pull the compression arm sleeve gradually over the elbow and up to the axilla.

TIP: Hold on to some support (e.g. a banister, door handle) with the arm to be dressed.

The compression arm sleeve fits correctly when it covers the arm evenly without forming creases.

Special notes:

- Caution: long fingernails and jewelry may damage the garments when putting them on. For easier application use rubber gloves or a donning aid.
- Do not bring the compression garments into contact with greasy ointments, body lotions or similar substances unless recommended by JOBST®.
- Loose threads or loops should not be cut off, since this may cause holes or runs to form in the garment.
- Intended to be used by a single patient only.

2. Care instructions

JOBST® compression garments should, if at all possible, be washed daily, or at least every other day.

For this purpose, you may use a regular mild detergent. Do not use laundry additives such as fabric softener, optical brightener or stain remover. Rinse your compression garment well.

Roll up the wet compression garments in a towel after washing and press out excess moisture. Do not wring. Do not dry the compression garment on a radiator or in direct sunlight.

Silicone band care:

When needed, hand wash silicone band to remove excess creams, lotions, hair and other foreign material.

3. Indications

Every JOBST® Confidence compression product has a specific compression level designed to help specific venous and lymphatic conditions.

The compression level is printed on the package label.

The required compression class must be determined by your physician.

Medical compression garments are used for the management of the following indications:

- Primary and secondary arm lymphedema in ISL stages I-II with no shape distortions
- Lipedema
- Paget-von-Schrötter syndrome (axillary phlebothrombosis)
- Klippel-Trenaunay symptom complex
- Hypertrophic scars

4. Contraindications

JOBST® Confidence compression garments should not to be worn in cases of:

- Advanced arterial insufficiency including ischemia
- Uncontrolled congestive heart failure
- Untreated septic phlebitis
- Phlegmasia cerulea dolens
- Incompatibility to fabric

5. Cautions

- Weeping dermatosis
- Cutaneous infections
- Open wounds must be covered with an appropriate dressing before the compression garment is put on
- Severely compromised skin sensibility and impaired sensitivity of the limb
- Advanced peripheral neuropathy
- Immobility (confinement to bed). Not recommended to wear during bed rest.

- Conditions in which increased venous and lymphatic return is not desired.
- Appropriate hygiene and skin care should be applied.
- Ill-fitting compression garments may cause constriction leading to e.g. skin necrosis, pressure ulcers and pressure related peripheral nerve damage.
- The attending physician decides whether compression in general can be applied in patients suffering from the following diseases: rheumatoid arthritis, complex regional pain syndrome (CRPS, M. Sudeck), malignant lymphedema, gangrene.
- Medical compression garments should only be used after consultation with your doctor or therapist and checked regularly by them on the appropriateness of your therapy, also considering age and health status.
- Patient measurements should be taken by a trained person to determine correct product fit.
- Patients should not wear a lower compression class in JOBST® Confidence than indicated in the prescription or worn previously.
- Patients with pronounced skin folds and shape distortions should not wear JOBST® Confidence.
- Changing to a compression garment of different fabric may lead to variations in edema management.
- When re-ordering a garment after 6 month we recommend to take the patients measurements again as the limb circumference might have changed.
- If you notice discomfort, pain, numbness, changes in skin color, appearance of new wounds or swelling during wear, stop using the product and contact your healthcare professional.
- Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent authority of your state.

6. Directions for storage/Guarantee

Made-to-measure products are products intended for immediate use and no expiry date for storage is indicated on these. Due to wear or slackening of the material, the medical efficacy can be guaranteed for a wearing period of six months. This assumes correct handling (e.g. product care, donning and doffing).

Please protect your compression garments from direct exposure to sunlight, heat and moisture. We recommend storage at room temperature.

If the disease persists after 6 months, please consult your attending physician concerning the possible need of a new garment.

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BSN-JOBST GmbH
Australian Distributor: BSN medical (Aust.) Pty Ltd